

Directions for using **DENTEMP® O.S.**

To Replace a Lost filling

(To re-cement loose caps or crowns see other side)

Read directions completely before starting. Follow directions carefully as you proceed.

1. Wash hands and dry thoroughly.
2. Rinse out cavity area carefully with warm water.
3. Do not dry; leave cavity moist.
4. Use applicator stick to remove a small amount of material from multi-use vial, form into a ball with your fingers and press firmly into cavity area (*Diagram 1*).
5. Bite down several times to insure proper bite.
6. REMOVE ANY EXCESS MATERIAL.
RINSE OUT MOUTH WELL WITH WATER.
7. Allow 1 hour to set before eating.
8. This product is for temporary use only. Consult your dentist as soon as possible.
9. Keep vial closed to maintain freshness of remaining material.



Any left over material may be re-used as long as the material is soft and pliable. Protecting the material from the air will prolong its usefulness. Keep vial closed to maintain freshness of remaining material. Once the material hardens it is no longer useable and must be discarded.

Ingredients: Dimethoxytetraethylene Glycol, Ethyl Methacrylate, Calcium Sulphate, Zinc Oxide, Barium Sulphate, Eugenol

Caution: Do not use if you are allergic to Eugenol. This product should not be used if throbbing pain or swelling in affected area exists. Consult your dentist immediately. This product is sold as a temporary remedy. You must see your dentist as soon as possible for permanent work to be performed. Failure to do so may cause infections and/or serious injury.

Keep out of the reach of children

**Any problems or questions? Call us at (800) 238-0220 or (845) 436-0011,
Mon.-Thurs. 9:00 AM to 5:00 PM, Fri. 9:00 AM to 3:30 PM ET.
E-Mail us at: custserv@majesticdrug.com or visit our website at
www.Dentemp.com**

Directions for using
DENTEMP® O.S.
To Re-cement Loose Caps or Crowns
(To replace lost fillings see other side)

Read directions completely before starting. Follow directions carefully as you proceed.

1. Wash hands and dry thoroughly.
2. Prepare crown by carefully removing as much old cement as possible from the inside of the crown. Rinse well.
3. Carefully clean and rinse tooth area with water and leave moist.
4. Try to fit the crown back on the tooth before applying Dentemp O.S. Make sure it fits properly. If you cannot do this, **DO NOT PROCEED.** See your dentist.
5. Use applicator stick to remove a small amount of material from multi-use vial. Place a thin layer of material (the thickness of 2-3 sheets of paper) on the upper inside wall of the crown (*Diagram 2*).
6. Remoisten tooth area, place crown on wet tooth and press firmly.
7. Bite down several time to insure proper fit. If the bite is not completely comfortable remove and repeat all steps until the crown feels normal.
8. **REMOVE ANY EXCESS MATERIAL. RINSE OUT MOUTH WELL WITH WATER.**
9. Allow at least 1 hour to set before eating.
10. This product is for temporary use only. Consult your dentist as soon as possible.
11. Keep vial closed to maintain freshness of remaining material.



Note: When replacing a bridge apply **Dentemp O.S.** sparingly to only 1 hollow tooth at each end of the bridge. **Dentemp O.S.** should not be used on implants.

Any left over material may be re-used as long as the material is soft and pliable. Protecting the material from the air will prolong its usefulness. Keep vial closed to maintain freshness of remaining material. Once the material hardens it is no longer useable and must be discarded.

Caution: Do not use if you are allergic to Eugenol. This product should not be used if throbbing pain or swelling in affected area exists. Consult your dentist immediately. This product is sold as a temporary remedy. You must see your dentist as soon as possible for permanent work to be performed. Failure to do so may cause infections and/or serious injury.

Keep out of the reach of children. See other side for contact information

©2003 Majestic Drug Co., Inc. P.O.Box 490, 4996 Main St. (Rt. 42), South Fallsburg, NY 12779 U.S.A.