

RECAPIT[®] CEMENT

Replace Loose Caps

Questions? **800-238-0220**

visit us at: www.majesticdrug.com

*Our guarantee: Follow easy directions and get best results available until you see your dentist.

Recapit[®] is a unique, no mix cement made especially for replacing loose caps and crowns. Clinically tested and guaranteed; **Recapit[®]** is safe and so effective "you can eat on it".

⚠️ READ DIRECTIONS CAREFULLY BEFORE USING **KEEP OUT OF REACH OF CHILDREN**

Caution: This product should not be used if throbbing pain or swelling in affected area exists. Consult your dentist immediately. This product is sold as a **temporary remedy**. See your dentist for permanent solutions.

Directions: **1)** Wash hands and dry thoroughly **2)** Prepare crown by carefully removing as much old cement as possible from the inside of the crown. Rinse and dry well. **3)** Carefully clean and rinse tooth area with water. Do not dry. Leave moist. **4)** Try to fit the crown back on the tooth before applying **Recapit[®]** cement. Make sure it fits properly. If you cannot do this, **DO NOT PROCEED**. See your dentist. **5)** Make sure inside of crown is dry. Use applicator brush to remove a small amount of material from multi-use vial. Paint a layer of material on the upper inside edge of the dry crown. **6)** Remoisten tooth area, place crown on wet tooth and press firmly. **7)** Bite down several times to insure proper fit. If the bite is not completely comfortable remove and repeat all steps until the crown feels normal. **8)** **WAIT 5 MINUTES AND THEN GENTLY REMOVE ANY EXCESS MATERIAL. RINSE OUT MOUTH WELL WITH WATER.** **9)** Allow at least 1 hour to set before eating. Cement will fully set in 1 to 3 hours. **10)** Clean brush thoroughly by rinsing in warm water and wiping with a paper towel or tissue. **11)** Keep vial closed to maintain freshness of remaining material. Note: When replacing a bridge apply **Recapit[®]** sparingly to only 1 hollow tooth at each end of the bridge. **Recapit[®]** should not be used on implants or posts without dentists advice. Any leftover material may be re-used as long as it remains soft and pliable. Once the material hardens it is no longer useable and must be discarded.

Ingredients: Calcium Sulphate, Dimethoxytetraethylene Glycol, Zinc Oxide, Potassium Sulphate, Ethylmethacrylate Polymer, Triacetin.

Authorized representative in EC: Comedic Int. Sales, 32A
Eveline Rd, Unit B, Mitcham, Surrey, England www.comedic.co.uk

© 2008 Majestic Drug Co., Inc., South Fallsburg, NY 12779



0 1070540070 8