

REFILIT[®]
FILLING MATERIAL *cherry*
Repairs
Lost Fillings

Questions? **800-238-0220**

visit us at: www.majesticdrug.com

*Our guarantee: Follow easy directions and get best results available until you see your dentist.

REFILIT[®] provides fast relief of pain and discomfort until you can see your dentist. Clinically tested, **REFILIT**[®] has a pleasant cherry taste, and is so safe and effective *"you can eat on it"*.

⚠ READ DIRECTIONS CAREFULLY BEFORE USING KEEP OUT OF REACH OF CHILDREN

Caution: This product should not be used if throbbing pain or swelling in affected area exists. Consult your dentist immediately. This product is sold as a **temporary remedy**. See your dentist as soon as possible for permanent work to be performed. Failure to do so may cause infections and/or serious injury. Wash hands before using.

To Replace a Lost filling: Wash hands thoroughly. Rinse out cavity with warm water. Do not dry; **leave moist**. Use applicator to remove a small amount of material from vial, form into a ball with your fingers and press firmly into cavity. Bite down several times to insure proper bite. **REMOVE ANY EXCESS MATERIAL. RINSE OUT MOUTH WELL WITH WATER.** Allow 1 hour to set before eating.

Keep multi-use vial closed to maintain freshness of remaining material. Any left over material may be re-used as long as the material is soft and pliable. Once the material hardens, it is no longer usable and must be discarded.

Ingredients: Calcium Sulphate, Barium Sulphate, Dimethoxytetraethylene Glycol, Zinc Oxide, Ethylmethacrylate Polymer, Cherry Flavour, Acesulfame-K.

Authorized representative in EC: Comedic Int. Sales,
32A Eveline Rd, Unit B, Mitcham, Surrey, England
www.comdic.co.uk

©2000 Majestic Drug Co., Inc., South Fallsburg, NY 12779



0

10705 40088

3